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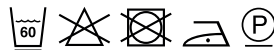
Your body shape determines the shape and use of a pillow, so what makes a pillow "right" is as individual as the person using it. Crucial criteria when selecting a pillow are a combination of the width of your shoulders, your weight, your mattress type and your sleeping habits and positions. The muscle relaxing effect results from relieving the pressure on the neck vertebrae. There should be no pressure especially where your neck vertebrae meet your shoulders, so that the discs are free of stress and can regenerate overnight.

As a general rule, a good pillow should not distort the spine in the neck area but should allow your head to rest straight and level, avoiding pressure on the discs in the neck area. To successfully achieve this, the pillow must be carefully selected or be able to be individually adjusted.

- Cover: 100% cotton (organic)
- Filling: 100% natural latex strips
- Amount of filling:
  - approx. 550 g bottom chamber
  - approx. 750 g top chamber
- Height: approx. 8 cm
- Size for covers: approx. 40 × 80 cm (cover not included)
- Made in Germany

### Product benefits

- Effective support for the lower cervical vertebrae (C5 + C6)
- Slight springiness provides relaxation for the neck muscles
- Super elastic thanks to filling of natural latex strips
- Particularly suitable for people who sleep on their sides or back
- Machine washable up to 60 °C in delicate mode
- Strictly tested for chemical harmful substances/residues

### Directions for use

#### Sleeping on your side (1)

If the pillow is too high when you sleep on your side, simply remove a portion of the filling material until the pillow is flat enough to let you rest in a level position.

#### Sleeping on your back (2)

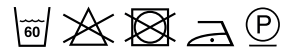
So that you are optimally supported when laying down, shake the pillow to move the filling in the top chamber to the sides. This way, your head will automatically be supported from both sides.

#### Individual adjustment (3)

Filling can be removed suit your individual requirements.

### Care instructions

The pillow is machine-washable up to 60 °C in delicate mode.

### Handy tips

If possible, air your pillow after each use so that all moisture can evaporate and the pillow feels fresh again.