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Anleitung
Instructions
Instructions

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Version: 2750_EL2_2111_Sz

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Federal Ministry of Health
Dept. Pharmaceuticals and Medical Devices
Radetzkystraße 2, 1030 Wien/Vienna
E-Mail: meddev@bmgf.gv.at

Für Schweiz

Swissmedic, Swiss Agency for Therapeutic Products
Medical Devices Division
Hallerstrasse 7, 3012 Bern, Switzerland

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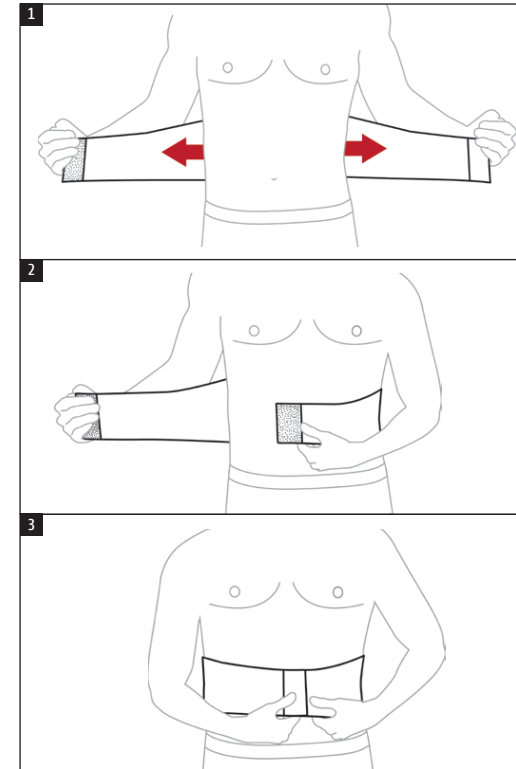
Bicycle support belt 458665

Version: 2750_EL2_2111_Sz

Scope of delivery:**1 x bicycle support belt**

Read the instructions for use completely and carefully!

**Purpose**

To activate and support the muscles of the abdomen and back through external pressure (compression). The belt supports the spine and promotes an upright posture.

- slipped discs
- sciatica pain
- lumbago
- pain from overstraining

For personal use

**Contraindications / Unsuitability**

In the case of the following pathologies, the bicycle support belt should only be used after consultation with your doctor:

- skin diseases/injuries in the affected part of the body, especially inflammatory conditions such as swelling, redness and overheating.
- sensory disturbances and movement disorders, e.g. with diabetes.
- open wounds or internal injuries
- allergic reactions to the material.

**Instructions for safe use of the product**

There are no known side effects that involve the entire organism. Proper use is a prerequisite. Product liability is excluded in the case of improper use.

- Immediately remove the bicycle support belt if it is too tight or if you find it uncomfortable!
- Wear the bicycle support belt only for an hour at a time, not permanently.
- Consult your doctor immediately if the symptoms worsen or you notice any changes in your condition.
- All serious incidents involving the product must be reported to the manufacturer and the responsible authorities of the Member State where the user and/or patient is located.

For Germany

Federal Institute for Drugs and Medical Devices
Medical Devices Division
Kurt-Georg-Kiesinger-Allee 3
53175 Bonn
<https://www2.bfarm.de/medprod/mpsv/>

For Austria

Federal Ministry of Health
Dept. Pharmaceuticals and Medical Devices
Radetzkystrasse 2, 1030 Vienna
Email: meddev@bmgf.gv.at

For Switzerland

Swissmedic, Swiss Agency for
Therapeutic Products
Medical Devices Division
Hallerstrasse 7, 3012 Bern, Switzerland

Do not use

- in case of infants or persons who would not notice if the bicycle support belt is placed uncomfortably.
- while sleeping.


Use / applying the belt

Put on the support belt (reflectors facing outwards, pocket opening facing upwards):

Fig. 1: Hold the support belt with the Velcro fastener (inside) in your right hand and guide it around your back. First pull the two straps away from the body at the side.

Fig. 2: First fold in the left side, then the right. The first Velcro fastener makes it easier to put on.

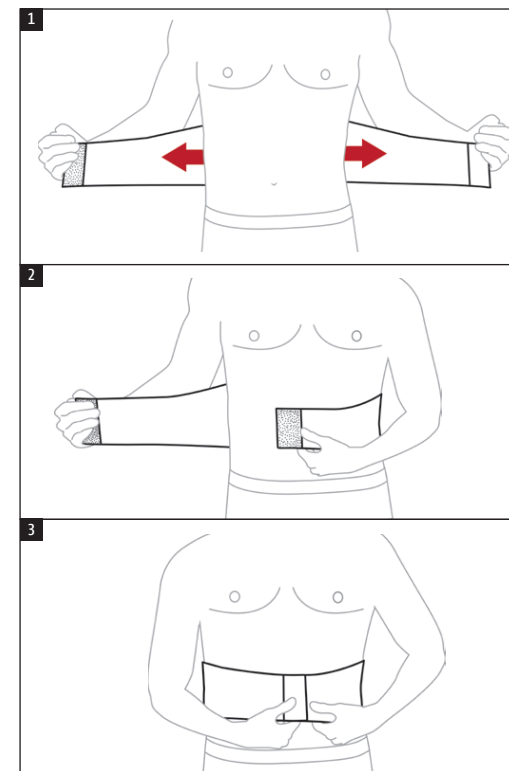Fig. 3: Then use light pressure to attach the second Velcro fastener to any point on the harness.

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Cleaning

Close the Velcro fasteners before washing to prevent damage to other items of laundry. It can be washed with the household laundry – see the care instructions.

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Biber Umweltprodukte Versand GmbH
HyGas GmbH & Co. KG, Hirzenhainer Straße 3,